


INSTRUMENT TRAY

INCLUDING MINI COMPUTER

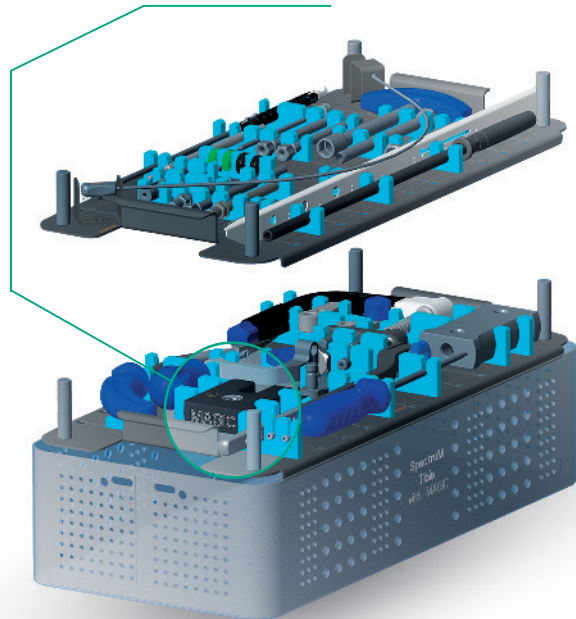

Maximum sterilization temperature: **135°C**



Power source enough for **1000** operations

WHY MAGIC?

- ✓ Shorter operation time
- ✓ More accurate targeting
- ✓ Less radiation load
- ✓ Less stress, longer life
- ✓ World-unique product
- ✓ Good business opportunity



100%
REDUCTION



100%
ACCURACY



25%
REDUCTION

MAGIC

FOR THE DISTAL TARGETING
OF THE INTRAMEDULLARY NAIL



LESS RADIATION

PERFECT AIMING?
NO X-RAY JUST A BIT OF MAGIC!


3300 Eger, Faiskola u. 5., Hungary
Phone: +36 36 512 900
E-mail: metal@sanatmetal.hu




REV_A_201611_ENG

MAGNETIC TARGETING NO RADIATION LOAD!

MAGIC consists of a magnetic signal source and a signal processing mini computer which first runs a calibration cycle mapping the neighbouring magnetic signals. After that the magnetic signal source is pushed into the intramedullary nail and the LEDs show the rotation direction of the distal arm and also the correct position of the special hole.

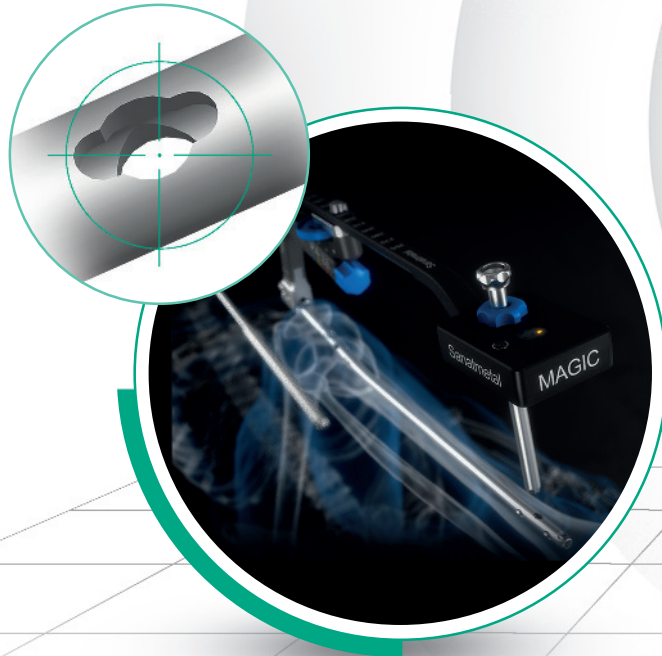

SPECIAL HOLE+ HAND TARGETING DEVICE= FASTER, MORE ACCURATE, NO RADIATION LOAD!

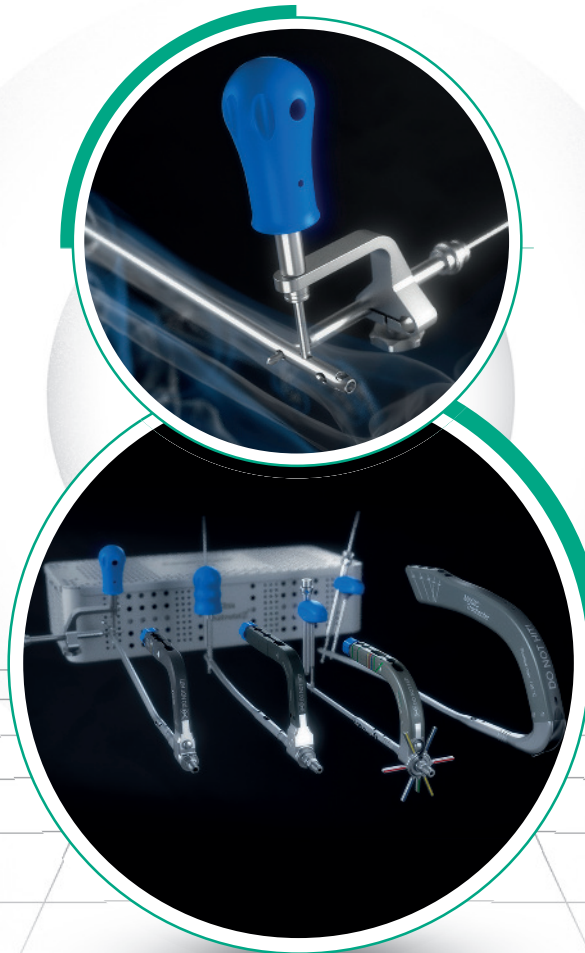

PUSH-OF-A-BUTTON ROTATING TARGETING ARM NO TARGETING ARM ASSEMBLY, REDUCED OPERATION TIME!

